elvie

pump

Instructions for Use

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Manuel d'instructions

Handleiding

Manual

elvie.com

Language	Page
English	4
简体中文	18
繁體中文	32
Español	46
Deutsch	60
Français	74
Nederlands	88
Svenska	102

Make sure you charge Elvie Pump and download the app.

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Instructions for Use

- Download the Elvie Pump app to make use of the smart features: monitor milk volume in real time, track sessions and control the pump remotely
- The app is available on devices that support Bluetooth Low Energy. iPhones must be iPhone 5S or later and have iOS10 or above. Android phones must have version 6 or newer
- For other languages, visit elvie.com/languages
- For the most up to date information, please check elvie.com

Questions?

For further assistance, please refer to the Help section in the Elvie Pump app or visit elvie.com. To find our Customer Care contact details for your region, please go to our website.

1. Intended use

1.1 Indications for use

Powered breast pump, to express and collect milk from lactating women. The device is intended for a single user.

1.2 Operating life

- The operating life of the Hub is defined to be approximately 500 hours
- The expected operating life of the replaceable washable components is six months
- It is recommended to replace the washable components after long-term storage or before pumping for your next child

2. Getting started

2.1 Taking apart Elvie Pump

1. Press the Bottle Release to separate the Bottle from the Hub

2. Remove the Valve from the Spout


3. Using the Spout, you can pour your expressed milk into a container of your choice for storage or feeding

4. Twist the Spout to remove it from the Bottle

5. Pull the Breast Shield away from the Hub to remove it

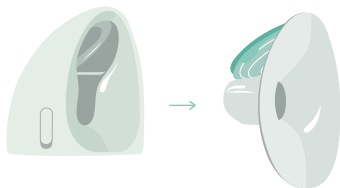6. Take the Seal off the Breast Shield

2. Getting started

2.2 Cleaning before first use

Elvie Pump is not provided sterile. It must be cleaned and sterilized before first use. Always take apart components before cleaning or sterilizing.

The Hub: electronics care


Do not get wet!

The Hub contains electronics that should be kept dry. A clean, damp cloth can be used to clean the Hub. The black semicircle on the base of the Hub monitors milk volume. Please keep this area dry and clear of residue. Any other cleaning method will be classed as misuse under the warranty (see section 4).

Washable components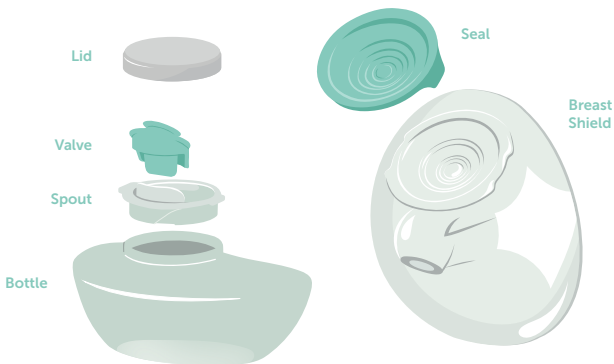

Cleaning washable components that come into contact with milk prevents the build-up of dry milk residue, which can cause the growth of bacteria that may be harmful to you and your baby.

2. Getting started

Before first use

Clean washable components

- 1. Rinse** in cold, clear water (approx +20°C / +68°F)
 - 2. Wash** in warm water (approx +30°C / +86°F) using mild dish washing soap and wipe afterwards with a clean cloth
 - 3. Rinse** with cold, clear water (approx +20°C / +68°F) for 10 to 15 seconds
-

OR

Place on the top shelf
of the dishwasher

Sterilize washable components

Boil in water for five minutes

OR

Suitable for microwave,
steam or cold water
sterilizing methods.
We do not recommend microwave
or steam sterilizing for more than
five minutes*

Dry completely before assembly

*You should always refer to the manufacturers instructions and please be aware that some steam and microwave sterilizers reach high temperatures which may risk warping plastic parts. We do not recommend ultraviolet sterilisation.

2. Getting started

2.3 Charging

1. When the battery is low, the Battery Light will be red.
2. When charging, the Battery Light will be orange and Elvie Pump will not turn on.
3. When fully charged, the Battery Light will be green.

Battery low

2.4 Breast Shield sizing

Elvie Pump comes with Breast Shields in two sizes, so you can find a Breast Shield fit that is comfortable for you. If needed, a smaller Breast Shield size is available through [elvie.com](https://www.elvie.com) or authorized retailers.

1. To determine the correct size, put the Breast Shield on with the sizing symbol facing upwards and place your nipple in the center of the fit lines. The fit lines should align with the outside of your nipple.
2. Try different Breast Shields to find the best fit for you.
3. To comfortably pump on both sides, repeat this process for both nipples to ensure you have the correct Breast Shield for each nipple.

Available as an accessory

○ 21mm

Breast Shield too small

Included in the box

△ 24mm

Breast Shield correct fit

□ 28mm

Breast Shield too big


Align the Breast Shield to the direction of your nipples when you are putting it on:

- Bend forwards and hold your breast from underneath
- Bring the Breast Shield to your breast, paying attention to the direction of your nipple
- You can use the markings on the Breast Shield to help align your nipple


2. Getting started

2.5 Assembling Elvie Pump

1. Make sure all parts are dry before putting them together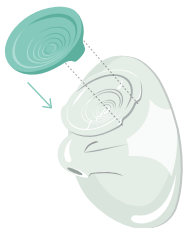


2. Push the Breast Shield into the Hub

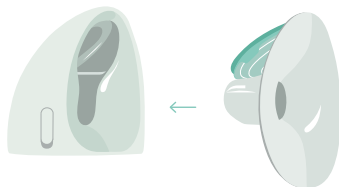3. Twist and click the Spout onto the Bottle

4. Push the Valve onto the Spout

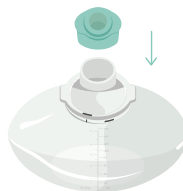5. Click the Bottle onto the Hub

3. Using Elvie Pump

3.1 Turning on Elvie Pump

Once fully assembled, press Power to switch Elvie Pump on. Play / Pause will flash.

3.2 Finding the right compression

Elvie Pump is designed to be worn with your standard nursing bra and you can use it in either a sitting or standing position. Getting the right compression could impact milk volumes. Elvie Pump needs to be firmly pressed against your breast, there should be no gaps between Elvie Pump and your breast. Every bra has a different level of support, so make sure to get the compression right before each session. If you are using a particularly structured and inflexible bra, the Bra Adjuster may be used to create more room. The Elvie logo on the Bra Adjuster should face outwards.

Tip: The Bra Adjusters are machine washable. To order additional Bra Adjusters, please visit elvie.com or an authorized retailer.

3.3 Starting your session

Once Elvie Pump is turned on and positioned in your bra, press Play / Pause to start pumping.

3.4 Intensity settings and modes

Modes

- Elvie Pump has two modes: Stimulation and Expression. Elvie Pump will always start in Stimulation mode. This is a faster pumping rhythm to stimulate milk let-down
- Elvie Pump will automatically switch to Expression mode either when it detects milk has started to flow or after approximately two minutes


Switching sides

- If you are switching Elvie Pump from one breast to the other, make sure you press the Side Selector while Elvie Pump is paused. When you restart Elvie Pump on your other breast, it will start in Stimulation mode

Intensity settings

- Elvie Pump has seven intensity settings, giving you control over what feels comfortable and works most efficiently in both Stimulation and Expression modes
- Once you can feel the suction on your nipple, use + - to adjust intensity to find your preferred setting in both modes. When you reach the maximum or minimum intensity setting, Play/Pause will flash three times

Note: Please visit elvie.com for the full list of automatic functions.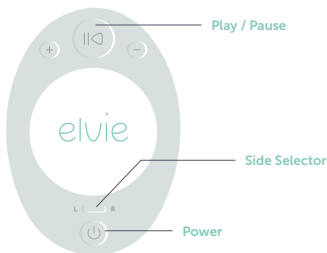


3. Using Elvie Pump

3.5 Changing the Bottle

Elvie Pump will automatically pause when the Bottle is full, indicated by the Bottle Light. You must empty the Bottle before resuming your session.

Press the Bottle Release to separate the full Bottle from the Hub.

Replace the full Bottle with an empty Bottle or pour your expressed milk into a container of your choice and continue with the same Bottle. Click the empty Bottle on to the Hub.

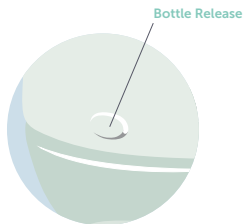3.6 Ending your session

1. Press Play / Pause to stop pumping.
2. Unclip your bra and use your finger to break the seal around your nipple to avoid nipple damage.
3. Remove Elvie Pump from your bra.
4. Press Power to switch off.

3.7 Cleaning

Clean washable components after each use as advised.

3.8 Storing your milk

You may store your expressed milk in the Bottle using the Lid provided (both are fridge and freezer safe). Always make sure you remove the Bottle from the Hub before storing.

3.9 How to store Elvie Pump

Short-term storage

Between sessions, please store your Elvie Pump out of direct sunlight and ensure the washable parts are dry. Keep out of reach of children. Recharge immediately after finishing your pumping session if the Battery Light is on.

Long-term storage

Please use the Charging Cable provided to charge Elvie Pump before long-term storage and store out of direct sunlight. These measures will increase the lifespan of your Elvie Pump. When removing from storage, plug the Charging Cable into your Elvie Pump and fully charge for approximately two hours.

The washable parts will require cleaning and drying before long-term storage and should be stored out of direct sunlight to avoid discoloration.

4. Warranty

Elvie Pump is backed by a limited manufacturer's warranty against any defects in materials and/or workmanship: the Hub for two (2) years and the washable components for 90 days from the date of original purchase. Defects discovered within the warranty period should be reported to Chiaro Technology Limited, "Chiaro".

This warranty is provided free of charge and is separate to any rights provided under consumer laws or regulations in the country of purchase. It does not exclude, limit or affect any consumer rights or legal claims arising from national laws on the sale of consumer products. This warranty is governed by and constructed under the laws in which the product purchase took place.

Chiaro reserves the right to a period of investigation to establish the cause of the defect and may exercise its discretion in determining whether or not to replace free of charge.

This warranty does not cover cosmetic deterioration or damage caused by general wear and tear, physical or natural destruction, accident, misuse, neglect or other external causes. Any attempt to take the Hub apart will invalidate this warranty.

The warranty does not cover purchases from non-authorized resellers.

Any warranty claims must be supported with reasonable evidence, including proof of date of purchase, alongside these warranty conditions. Chiaro must be notified of any defects. We will then provide you with a Return Merchandise Authorization number, which must be displayed on the outside packaging. In the event that Chiaro decides to provide a replacement, any replacement will not extend the life of this warranty.

This is a single-user product. Use by more than one person may present a health risk and voids the warranty.

Please keep your proof of purchase in the event that you need to contact customer care.

5. Product specifications

5.1 Technical specifications

Vacuum range:	40 to 220 mmHg
Power:	5v DC, 3A max
Battery capacity and type:	1460mAh Lithium polymer
Size:	128mm x 110mm x 68mm / 5in x 4.3in x 2.7in
Weight:	225g / 7.9oz
Bluetooth info:	Bluetooth v5 (Bluetooth Low Energy). Range up to 10m
Operation temperature:	+5 to +30°C / +41 to +86°F
Transport/storage temperature:	-20 to +60°C / -4 to +140°F
Operation humidity:	15 to 90%
Transport/storage humidity:	15 to 90%
Ambient pressure:	90 to 106 kPa
Infrared details:	Peak frequency: 319THz, Radiant power: 55mW

5.2 Transport, storage and disposal

Transport

While traveling, use the Charging Cable provided and pair with the appropriate power source for your current location. Please check with the airline prior to travel for advice on carrying and using Elvie Pump while flying.

Temperature: -20 to +60°C / -4 to +140°F

Relative humidity: 15 to 90%

Storage

Temperature: -20 to +60°C / -4 to +140°F

Relative humidity: 15 to 90%

Ambient pressure: 90 to 106 kPa

Disposal

Elvie Pump contains a rechargeable lithium polymer battery. Please dispose of the Hub at a designated collection point for the recycling of waste electrical and electronic equipment.

The appropriate disposal of Elvie Pump will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste for recycling, please contact your local authority or where you purchased your product.

5. Product specifications

5.3 EMC advice

Declaration for Electromagnetic Compatibility (EMC): For information use only.

FCC Statement: This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the equipment and receiver
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected
- Consult the dealer or an experienced radio/TV technician for help

Stacking: This device should not be used adjacent to or stacked with other equipment. And if adjacent or stacked use is necessary, this device should be observed to verify normal operation in the configuration in which it will be used.

FCC Radiation Exposure Statement: This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Caution: The user is cautioned that changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Manufactured and Distributed by Chiaro Technology Limited

Model number: EP01

Document number: EP-S-0005

Revision number: 10

6. Safety

6.1 Warnings

- This is a single-user product. Use by more than one person may present a health risk and voids the warranty
- It is recommended you consult with a licensed healthcare professional before using Elvie Pump if you have an active implanted device that would be affected by a magnetic field (e.g. pacemaker, implantable cardioverter defibrillator)
- Inspect for damage, clean and sterilize washable components that come into contact with your breast and breast milk prior to first use
- Clean all parts that come into contact with your breast and breast milk, and inspect for damage after every use
- During normal use, areas of the Elvie Pump may warm up. If your pumping environment is at the upper operating temperature limit (+30°C / +86°F) for the maximum session time (40 minutes), the temperature of the Breast Shield may reach up to +43°C / +109°F
- The black semicircle on the base of the Hub uses infrared (IR) energy to monitor the milk volume in the bottle when Elvie Pump is pumping. Ensure the black semicircle does not come into direct contact with skin while Elvie Pump is pumping
- Do not store wet or damp parts, as mold may develop
- Do not immerse the Hub in water or other liquids. Wipe with a clean, damp cloth
- This product contains small parts and a cable that may present a hazard. Keep out of reach of children
- Only use Elvie manufactured parts with Elvie Pump, including the Charging Cable
- It is recommended to use an adaptor compatible with both IEC 60601-1 and your regional power source
- Do not drop or insert any foreign object into any openings on the Hub and always close the Charging Cap after charging
- Never operate an Elvie Pump if it has been damaged, is not working properly, or has been dropped into water. If damage is found, immediately discontinue use
- If the package appears damaged or tampered with before use, do not use and contact our Customer Care team
- Elvie Pump and its components are not heat-resistant. Keep away from heat sources
- Do not use flammable materials while using Elvie Pump or connecting Elvie Pump to a charger
- Elvie Pump should not be used while lying down or engaging in strenuous exercise or activities with risk of harm
- Do not use Elvie Pump while sleeping or overly drowsy
- Do not wear Elvie Pump when not pumping
- Do not attempt to reset Elvie Pump during use
- Do not use Elvie Pump when driving or operating heavy machinery

- Elvie Pump uses Bluetooth technology. Please follow airline rules for the use of portable electronic devices when flying. Furthermore, high altitudes, including while in a pressurized aircraft cabin, will affect the performance of Elvie Pump
- Always break the seal around your nipple before removing Elvie Pump from your breast
- Do not pump until after giving birth as breast pumping can induce labor
- Portable and mobile frequency communications equipment can affect the breast pump
- Only use Elvie Pump for its intended use as described in this manual


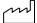
6.2 Precautions

- Elvie Pump will not work when connected to the charger; always disconnect Elvie Pump from the charger and close port
- Always ensure all parts are completely dry before assembling
- Do not place the Hub in the fridge or freezer
- Contact your healthcare professional if you express minimal or no milk, or if expression is painful
- For assistance with correct Breast Shield sizing and comfort, please see section 2.4 or talk to a lactation consultant or breastfeeding specialist
- Discontinue use if you notice a loss of power or if the fully charged battery lasts for less than two hours of continuous use. The battery is not replaceable
- Wash your hands thoroughly with soap and water before touching Elvie Pump, its components or your breasts, and avoid touching the inside of the Bottle or Lid
- Only use drinking-quality tap water or bottled water for cleaning components that come into contact with milk
- Do not use harsh chemicals to clean the Breast Shield, Bottle, Valve, Spout or Seal
- If stored at minimum or maximum storage temperatures (see section 5.2), when removing from storage, please allow 24 hours for the pump to reach normal operating temperatures before use
- Do not expose Elvie Pump or its components to direct sunlight

Note: Please visit [elvie.com](https://www.elvie.com) for the full list of troubleshooting including Alerts, warnings and factory reset for Elvie Pump

6. Safety

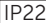6.3 Symbols

	On/Off
	Increase intensity
	Decrease intensity
	Left and Right side selection
	Play/Pause
	Consult instructions for use
	Manufacturer
	Date of manufacture
	Do not dispose the device together with unsorted municipal waste (in accordance with local regulations)
	Compliance with international requirements for protection from electric shock. (Type BF applied parts)
	Protection against entrance of solid foreign objects and against harmful effects due to the entrance of water
	Manufacturer has taken all necessary measures to ensure that the product complies with applicable safety legislation for distribution within the European Community
	Serial Number
	Lot Number

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