
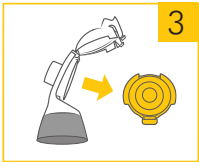
EN Cleaning

How to take apart

After each use

Disassemble the individual parts (breast shield, connector and breast milk bottle) as follows: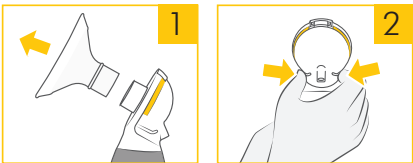


1. Remove the breast shield from the connector.
2. Open the back cap of the connector by squeezing both flaps and swiveling the cap upwards.
3. Remove the membrane from the connector body.
4. Separate the breast milk bottles and lids.

How to wash (before first use and after each use)

Washing is important for hygiene and serves to clean the surfaces of the parts by physically removing contamination. Wash the parts either by hand or in a dishwasher. For additional cleaning guidelines see the Center for Disease Control website.


Do not place the parts directly in the kitchen sink for rinsing and washing. Use a dedicated wash basin for infant feeding items.

Washing in a dedicated basin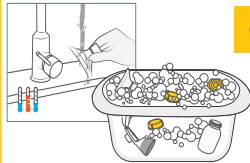

or

Wash in dishwasher

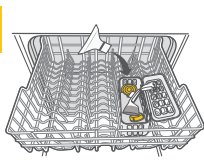1. Rinse the disassembled parts, except for the tubing, with cold, clear drinking-quality water.
 2. Soak with warm soapy water for 5 minutes and wash with a clean, unused dish cloth. Use a commercially available dish soap, preferably without artificial fragrances and coloring (pH neutral).
 3. Rinse the parts with cold, clear drinking-quality water.
 4. Store dry parts when not in use. Do NOT store wet or damp parts.
1. Place the disassembled parts, except for the tubing, on the top rack or in the cutlery section. Use a commercially available dishwashing detergent.
 2. Store dry parts when not in use. Do NOT store wet or damp parts.

Useful information

- If using the dishwasher, parts may become discolored. This will not impact the function of the parts.

Cleaning the breast pump (as needed)

1. Turn the breast pump off.
2. Unplug the breast pump from the power source.
3. Wipe the breast pump with a clean, damp cloth and dry with a clean towel.

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EN Quick Start Instructions

Before you pump for the first time, read your breast pump instructions for use.

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Medela wordmark and logo and Pump In Style are registered in the U.S. Patent and Trademark Office and the Canadian Intellectual Property Office. The Science of Care, Turning Science into Care and PersonalFit are trademarks of Medela.

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<https://bit.ly/MedelaFamilyDownload>

Breast Shield Sizing Guide

Choosing the right breast shield size can impact your breast milk supply. Find out why and get help choosing the right size.

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How to disinfect

(before first use and once per day)

Disinfecting is important for hygiene and serves to kill living organisms, such as bacteria or viruses. Boil the parts either on the stovetop or use Quick Clean™ microwave bags.

Disinfect by boiling

10min

Disinfect in the microwave

1. Cover the disassembled parts, except for the tubing, with water and boil for 10 minutes.
 2. Allow water to cool and gently remove parts from water with tongs.
 3. Place parts on a clean surface and / or towel and allow parts to air dry.
 4. Store dry parts when not in use. Do NOT store wet or damp parts.
1. Use Quick Clean* bags in the microwave in accordance with the instructions on the bags. (sold separately)
 2. Place parts on a clean surface and / or towel and allow parts to air dry.
 3. Store dry parts when not in use. Do NOT store wet or damp parts.
- Do NOT clean tubing in a micro-ste-am bag.

* Refer to local website/shops for availability in your country.


NOTICE

Pump In Style® Pro breast pump has a closed system so milk cannot get into the tubing when the pump is used as instructed.

Do not clean or rinse the tubing under any circumstance.

- Inspect the tubing before use. If you find condensation, breast milk or mold in the tubing, discontinue use and replace tubing.
- Do not run your breast pump with wet tubing; doing so will cause damage to your breast pump.

Do not clean or rinse the tubing under any circumstance.

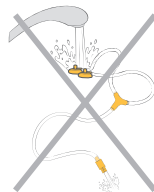How to Use Your Medela Pump in Style® Pro Video

Visit our website for helpful videos.

medela.qrd.by/pnspro-video-en

EN Quick Start

- 1 **Wash** breast pump kit parts – before first use and after each use
- 2 **Disinfect** breast pump kit parts – before first use and once per day
- 3 **Assemble** dry breast pump kit parts
- 4 **Insert tubing** - single (4a) or double (4b) pumping
- 5 **Power** your breast pump

Things you should know:

- If you find condensation, breast milk or mold in the tubing, discontinue use and replace tubing.

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15400289 A 0125 © 2025 Medela 101047267