MOM INSPIRED. HOSPITAL TRUSTED.

Purely Yours[®]

Breast Pump

Purely Yours Breast Pump Instructions

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Contents

Important Safeguards	1	Pumping Tips	8
Intended Use	1	<i>Pumping Time</i>	8
Before Every Use	1	<i>Checking Breast Flange Fit</i>	8
.....		Troubleshooting the Purely Yours Breast Pump	9
Purely Yours Breast Pump Diagram	2	Troubleshooting the HygieniKit Milk Collection System	9
.....		
Ameda HygieniKit® Milk Collection System Diagram	3	Ameda One-Hand Manual Breast Pump	10
.....		Set Up	10
Ameda® One-Hand Manual Breast Pump Diagram	3	<i>Before First Use</i>	10
.....		<i>After Every Use</i>	10
Replacement Parts and Accessories	4	Assembly	11
.....		<i>Converting from Electric Pumping to Manual Pumping</i>	11
Purely Yours Breast Pump	5	To Begin Pumping	12
Set Up	5	<i>Converting from Manual Pumping to Electric Pumping</i>	12
<i>Pump</i>	5	Pumping Tips	13
<i>Power Options</i>	5	<i>Pumping Time</i>	13
<i>Before First Use</i>	6	<i>Checking Breast Flange Fit</i>	13
<i>After Every Use</i>	6	Troubleshooting the One-Hand Manual Breast Pump	13
<i>Cleaning Pump</i>	6	
<i>Cleaning HygieniKit</i>	6	Technical Data	14
Assembly	7	Power Requirements	14
<i>Single and Dual Pumping</i>	7	Regulatory Compliance	14
To Begin Pumping	7	
<i>Starting Pump</i>	7	Product Disposal	14
<i>Setting Suction</i>	8	
<i>Setting Speed</i>	8	Warranty	15
<i>Turning Pump Off</i>	8	

IMPORTANT SAFEGUARDS

When using electrical products, especially if children are present, the following basic safety precautions should always be maintained.

READ ALL INSTRUCTIONS BEFORE USING

Danger: To reduce the risk of electrocution:

- Always unplug electrical devices immediately after use.
- Do not use while bathing, showering or swimming.
- Do not place or store where product can fall or be pulled into bathtub, sink or pool.
- Do not place or drop product into water or other liquid.
- Do not reach for electrical product that has fallen into water. Unplug from wall outlet immediately.

Warning: To reduce the risk of burns, electrocution, fire, or injury to persons:

- Never leave product unattended when plugged into electrical outlet.
- Close supervision is necessary when product is used near children or invalids.
- Use product only for intended use as described in this manual.
- Do not use attachments other than those recommended by manufacturer.
- Never operate if product has damaged cord or plug, is not working properly, or has been dropped, damaged, or becomes wet.

- Operate Purely Yours Breast Pump from batteries or car adapter (may be included; also sold separately) if reliability of electrical outlet is in doubt.
- Keep cord and all attachments away from heated surfaces.
- Never use while sleeping or drowsy.
- Do not use outdoors with cord.
- Do not operate where aerosol spray products are being used or oxygen is being administered.
- Batteries should be removed from pump when using the AC or car adapter.

SAVE THESE INSTRUCTIONS

For Your Health and Safety

The Ameda Purely Yours Breast Pump is a personal care item and should never be shared between mothers or resold or given from one mother to another.

Intended Use

The Ameda Purely Yours Breast Pump is intended to facilitate the expression and collection of breast milk from a lactating woman for the purpose of feeding the collected milk to a child.

Before Every Use

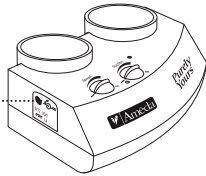
 Always wash hands well with soap and water before handling the Purely Yours, One-Hand Manual Breast Pump, HygieniKit Milk Collection System, or other Ameda products. Do not use solvents or abrasives.

Purely Yours Breast Pump Diagram
Diagramme du tire-lait Purely Yours
Diagrama de la bomba de extracción Purely Yours

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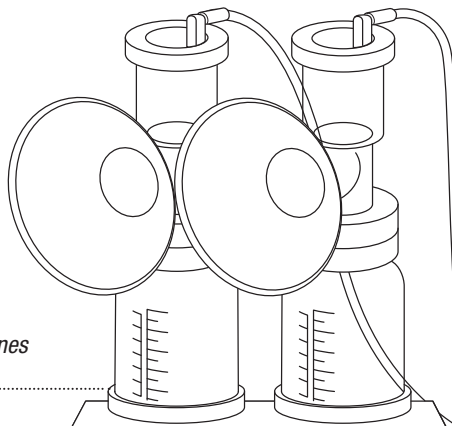
*Luz indicadora
de corriente*

Speed and Suction Dials

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AA Battery Compartment (underneath pump)

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

Replacement Parts and Accessories

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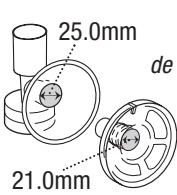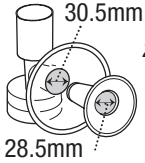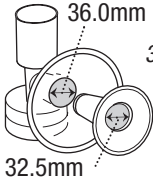
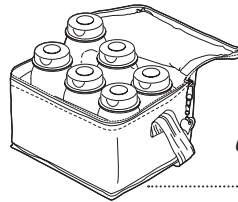
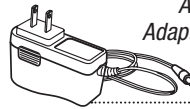
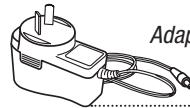
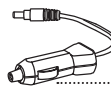
Purely Yours Breast Pump Set Up

ALWAYS WASH HANDS WELL WITH SOAP AND WATER BEFORE PUMPING.

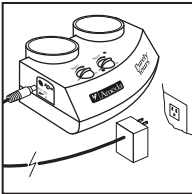
NOTE: Before using the Purely Yours Breast Pump and HygieneKit system, check the parts for cracks, chips, tears, color changes, or wear. They can affect the function of the kit. If parts need replacing due to defects, please call your local distributor or the location where you purchased the pump. For a listing of distributors in your country, please visit www.ameda.com.

Pump

Remove pump motor from box or case. Choose power option.


Power Options

The Purely Yours Breast Pump has three power options:


AC Adapter (included)

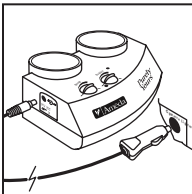Use only the AC adapter recommended for the Purely Yours Breast Pump.

Car Adapter (may be included; also sold separately)

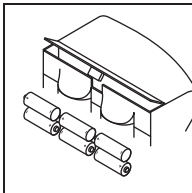Use only with the car adapter recommended for the Purely Yours Breast Pump.

6 AA Alkaline or Rechargeable Batteries (not included)

- Remove batteries if pump will not be in use for a long time. Dispose of batteries properly.
- When batteries are inside pump and AC adapter or car adapter is plugged in, pump will not draw power from batteries.

Warning: Batteries should be removed from pump when using the AC or Car adapter.

ALWAYS WASH HANDS WELL WITH SOAP AND WATER BEFORE PUMPING.

Before First Use

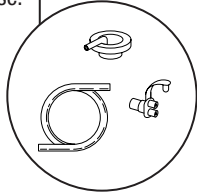
If your HygieniKit system is marked “STERILE,” open and use.

If your HygieniKit system is **NOT** marked “STERILE,” before first use:

- Boil all parts—except tubing, adapter cap, and tubing adapter—for 20 minutes.
- Remove all parts from water right after boiling.
- Allow parts to air dry before assembly and use.

Do not boil or wash tubing, adapter cap, and tubing adapter.

NOTE: Before using the HygieniKit system, check the parts for cracks, chips, tears, color changes, or wear. They can affect the function of the kit. If parts need replacing due to defects, please call:

In US: 1.866.99.AMEDA (26332)

In Canada: 1.800.604.6225

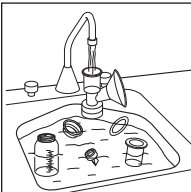
In other countries: Call your local distributor or the location where you purchased your pump. For a listing of distributors in your country, please visit www.ameda.com.

After Every Use

Cleaning Pump


Wipe the Purely Yours Breast Pump with clean, damp cloth.

Cleaning HygieniKit

- Take apart HygieniKit system to clean.
- Rinse any milk residue from pump parts with cool water.
- Wash gently by hand—except tubing, adapter cap, and tubing adapter—in warm, soapy water.
- Rinse well with warm, clean water.

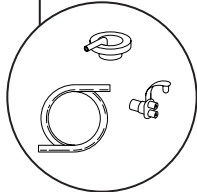
Do not boil or wash tubing, adapter cap, and tubing adapter.


Do not insert anything into white valve while cleaning.

- Allow parts to air dry on clean surface.

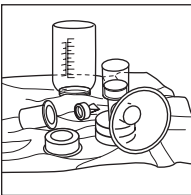Large plastic pieces (breast flanges and bottles) can be washed on the upper rack of a standard household dishwasher.


Assembly

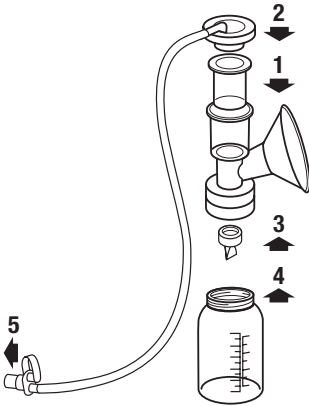
1. Insert diaphragm into upper chamber of breast flange.
2. Snap adapter cap with tubing onto the top of upper chamber.
3. Gently push white valve firmly onto ring on underside of breast flange.
4. Screw bottle onto breast flange.
5. Push tubing adapter at end of tubing into tubing adapter port on pump.

Single and Dual Pumping

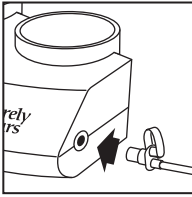The kit can be used to single or dual pump.

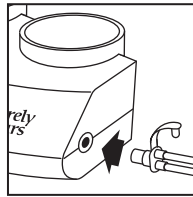To single pump:
Remove one tube and close tubing adapter.

To dual pump:
Both tubes should be attached to tubing adapter.

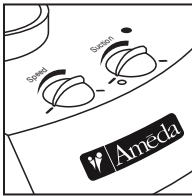
Insert the tubing adapter into the tubing adapter port of the pump.

To Begin Pumping

- Center nipple(s) in breast flange(s).
- Fill breast flange(s) with the breast(s) to create an air seal.


Starting Pump

- Choose power option, and connect pump to power source.
- Turn SUCTION dial to the right until light turns on. Pump is now on.

Turn dials to the right (clockwise) to increase pump settings (faster SPEED, stronger SUCTION). Turn dials to the left (counter-clockwise) to decrease pump settings (slower SPEED, gentler SUCTION).

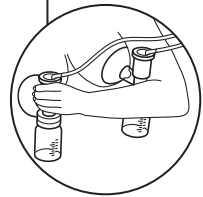Dual pumping with one hand gives you a free hand for other things.

Setting Suction

- Begin pumping at the highest comfortable SUCTION setting for you.
- Increase suction by turning SUCTION dial to the right (clockwise) until reaching maximum comfort level.
- If discomfort is felt in nipples(s) or breast(s), decrease SUCTION. If discomfort is felt at all suction settings, turn pump off, insert a finger between breast(s) and breast flange(s) to break suction and remove breast flange(s) from breast(s). Discontinue pumping and contact a healthcare provider.

Setting Speed

- Start with SPEED dial turned clockwise to the fastest setting.
- Once milk is flowing (let-down), turn SPEED dial counter-clockwise to a slower setting to drain the breasts faster.
- When milk flow slows turn SPEED dial clockwise to help trigger another let-down.
- At next let-down, turn SPEED dial counter-clockwise to a slower setting.
- Repeat steps as milk flow changes to pump more milk.

Turning Pump Off

- When finished pumping, turn SUCTION dial all the way counter-clockwise to turn pump off.
- After pump is off, if necessary, insert a finger between breast(s) and breast flange(s) to break suction.
- Remove breast flanges(s) from breast(s).

Pumping Tips

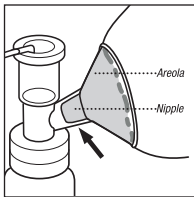
Pumping Time

Talk to your healthcare provider about how long you should pump. In most cases, 10 to 15 minutes per breast is suggested.

Checking Breast Flange Fit


The standard-size (25.0mm) breast flange fits some mothers well. If pumping causes discomfort, even on a low suction setting, you may need a different breast flange or insert size. A good breast flange fit may make pumping more comfortable and may express more milk.

To check your breast flange fit, watch your nipple during pumping. Use the pictures below as a guide.


Good Fit

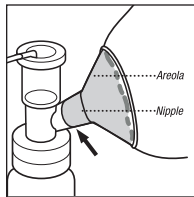During pumping, your nipple moves freely in the breast flange tunnel. You see space around the nipple. Not much areola is drawn into the tunnel with the nipple.

Too Small

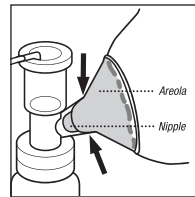During pumping, some or all of your nipple rubs against the sides of the breast flange tunnel.

Too Large

During pumping, more areola is drawn into the breast flange with your nipple. Your areola may rub against the side of the breast flange tunnel.

Visit www.ameda.com for more information.

Troubleshooting the Purely Yours Breast Pump

To check pump suction from motor:

- Remove tubing adapter from tubing adapter port of pump.
- Put a finger over tubing adapter port.
- If suction is felt, pump motor is working properly.
- If suction is not felt, unplug AC adapter, wait a few seconds, plug back into electrical outlet.

The Purely Yours Breast Pump has a reset feature. If the Purely Yours Breast Pump motor stops running:

- Unplug AC adapter cord from electrical outlet.
- Wait a few seconds and then plug AC adapter back into outlet.

Troubleshooting the HygieniKit Milk Collection System

If milk is dripping too slowly into bottle:

- Set SPEED dial to a faster setting.
- Replace silicone diaphragm(s) and white valve(s).

If pumping causes discomfort:

- Set pump to lower SUCTION setting.
- Check breast flange fit. A larger/smaller breast flange or insert may be needed.

If suction is low:

- Check that white valve(s) is/are pushed onto breast flange(s).
- Check white valve(s) for tears or stretches. Replace if needed.
- If single pumping, remove one tube and close one side of the tubing adapter.
- Check tubing adapter for cracks. If seen, replace.
- Set pump to higher SUCTION setting.

The white valves are vital for proper suction. To work well, they must be intact and look closed when pump is not in use. To prolong white valve life, follow cleaning instructions.

Ameda One-Hand Manual Breast Pump

Note: One-Hand Handle Assembly included with some versions. Also sold separately.

Set Up

ALWAYS WASH HANDS WELL WITH SOAP AND WATER BEFORE PUMPING.

Before First Use

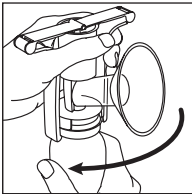
If your One-Hand Manual Breast Pump is marked “STERILE,” open and use.

If your One-Hand Manual Breast Pump is **NOT** marked “STERILE,” before first use:


- Boil all parts for 20 minutes.
- Remove all parts from water right after boiling.
- Allow parts to air dry before assembly and use.

After Every Use


- 1.** Take apart One-Hand Manual Breast Pump.

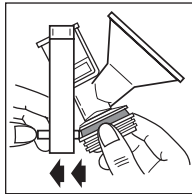
Unscrew bottle from breast flange.

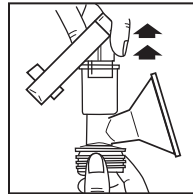
- 2.** Slide handle bottom off breast flange.

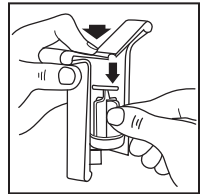
- 3.** Remove piston from upper chamber.

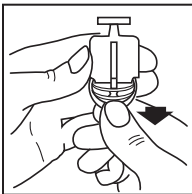- 4.** Push down on handle to release piston.

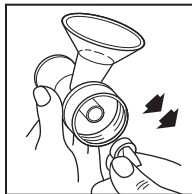
- 5.** Peel off piston seal from piston.

- 6.** Pull off white valve from breast flange.

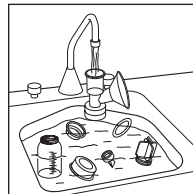
Note: Grasp white valve gently by thick sides, **not** by thin edge to prevent damage.

- 7.** Rinse any milk residue from pump parts with cool water.

Wash all parts in warm, soapy water.

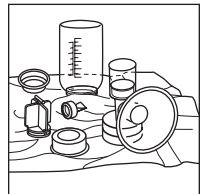Rinse well in warm, clean water.


- 8.** Allow parts to air dry on clean surface.


Large plastic pieces (breast flanges, handle, and bottles) can be washed on the upper rack of a standard household dishwasher. Wash piston, piston seal and white valve gently by hand. **Do not insert anything into white valve while cleaning.**

Assembly

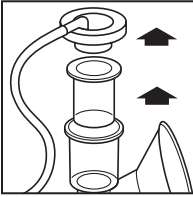
Converting from Electric Pumping to Manual Pumping

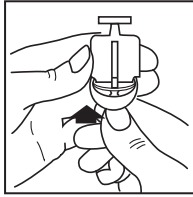
1. Unsnap and remove adapter cap.
2. Remove diaphragm from upper chamber of breast flange.

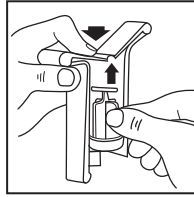
3. Fit piston seal onto base of piston.

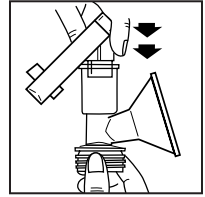
4. Push center of handle down and snap top of piston into groove on handle.

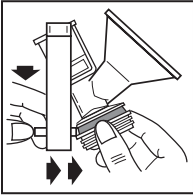
5. Push piston into upper chamber of breast flange.

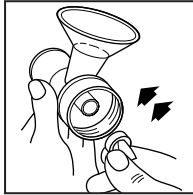6. Pull handle down and snap handle collar around groove of breast flange.
(See shaded area in diagram above.)

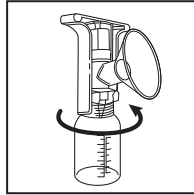
7. Gently push white valve firmly onto ring on underside of breast flange.


8. Screw breast flange onto bottle.
Handle is on correctly if top of handle is a straight line.

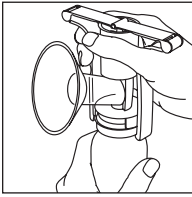
To Begin Pumping

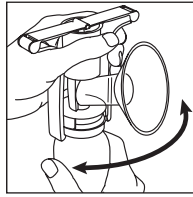
1. Grasp handle with thumb in front and fingers behind the upper chamber of breast flange.

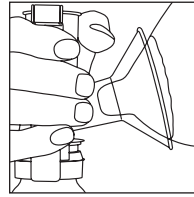
2. Swivel handle for wrist comfort.

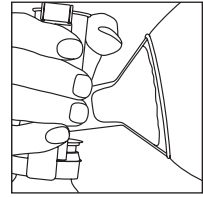3. Center nipple in breast flange.

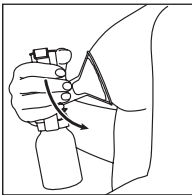4. Hold breast flange to the breast to make an air seal.


5. Tilt breast flange slightly downward so milk flows into bottle.


Watch your milk flow as you pump. Use it as your guide.

- Use a few minutes of fast, light squeezes to trigger milk let-down.
- When milk flows, use a squeeze, hold and release motion to pump milk or slow, strong but comfortable squeezes.
- When milk flow slows, use faster squeezes to help trigger another let-down.
- Repeat above steps as milk flow changes.
- If discomfort is felt during pumping, insert a finger between breast and breast flange to break suction and remove breast flange from breast. Discontinue pumping and contact a healthcare provider.

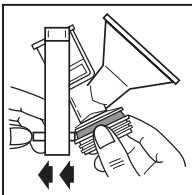Converting from Manual Pumping to Electric Pumping

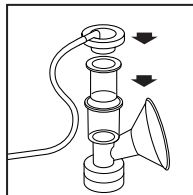1. Detach handle collar from breast flange and lift piston out of upper chamber.


2. Place diaphragm into upper chamber.


3. Snap adapter cap with tubing onto upper chamber.

4. Insert tubing adapter into tubing adapter port on the Purely Yours electric breast pump.

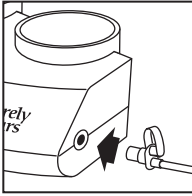The kit can be used to single or dual pump.

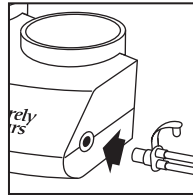To single pump:
Remove one tube
and close tubing
adapter.

To dual pump:
Both tubes should
be attached to
tubing adapter.

Pumping Tips

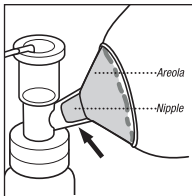
Pumping Time

Talk to your healthcare provider about how long you should pump. In most cases, 10 to 15 minutes per breast is suggested.

Checking Breast Flange Fit


The standard-size (25.0mm) breast flange fits some mothers well. If pumping causes discomfort, even on a low suction setting, you may need a different breast flange or insert size. A good breast flange fit may make pumping more comfortable and may express more milk.

To check your breast flange fit, watch your nipple during pumping. Use the pictures below as a guide.


Good Fit

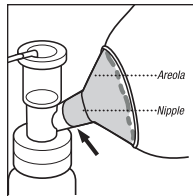During pumping, your nipple moves freely in the breast flange tunnel. You see space around the nipple. Not much areola is drawn into the tunnel with the nipple.

Too Small

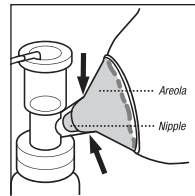During pumping, some or all of your nipple rubs against the sides of the breast flange tunnel.

Too Large

During pumping, more areola is drawn into the breast flange with your nipple. Your areola may rub against the side of the breast flange tunnel.

Visit www.ameda.com for more information.

Troubleshooting the One-Hand Manual Breast Pump

If milk is not flowing fast enough, adjust squeeze, hold, release frequency.

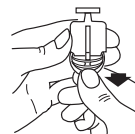
If pumping causes discomfort:

- Use gentler handle squeezes.
- Check breast flange fit. A larger/smaller breast flange or insert may be needed.

If suction is low:

- Check that white valve is fully pushed on to the breast flange.
- Check white valve for tears or stretches. Replace if needed.
- Remove piston seal, stretch out on flat surface, and replace on piston.

Peel Off Piston
Seal from Piston

Stretch
Piston Seal

Technical Data

Power Requirements

AC Adapter (included)

AC Adapter – North American Plug Configuration
9V/1.3A, 100-240V, 50/60Hz, direct plug in AC power adapter.

AC Adapter – Australia, United Kingdom, Europe Plug Configuration
9VDC/1.0A, 100-240V, 50/60Hz, direct plug in AC power adapter.

Car Adapter (may be included; also sold separately)

12VDC, F1.5A L250V fused adapter.

Batteries (not included)

Six AA batteries, 1.5 V alkaline or 1.2 V rechargeable

Technical Information

Operational conditions 

Transport and Storage conditions 

Fragile 

Protect from sunlight 

Keep Dry 

Regulatory Compliance

Regulatory: CE 0086

AC Adapters: UL, CSA

Motor Unit: UL E189700, CSA LR76525

Product Disposal

This product contains electrical and electronic equipment.

1. The internal components of this product may contain hazardous materials. Dispose of this product according to local or regional waste administration systems and regulations.
2. Do not dispose of as unsorted municipal waste.
3. For further information regarding the environmental performance of this product please visit our website www.ameda.com.

Warranty

The Ameda Purely Yours Electric Breast Pump (“Product”) is warranted to the original user — whether institutional or individual (“First User”) — only.

From the date of the First User’s purchase of this product, Ameda Company, Inc. (“Ameda”) warrants the Product to the First User against defects in material or workmanship for a period of one year for the pump mechanism and a period of 90 days on all other pump components. Ameda’s sole obligation under this limited warranty shall be to repair or replace, at Ameda’s option, any Product that is determined to be defective by Ameda and determined to be covered by this express limited warranty. Repair or replacement under this limited warranty is the sole and exclusive remedy of the First User. **Proof of purchase in the form of a receipted invoice or bill of sale evidencing that the Product is within the warranty period must be presented to obtain warranty service.** This limited warranty is extended by Ameda ONLY to the First User and is not assignable or transferable. For warranty service contact Ameda at 1.866.99.AMEDA (26332).

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THE PRODUCT IS HEREBY DISCLAIMED. AMEDA SHALL NOT BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL OR PUNITIVE DAMAGES OF ANY KIND FOR BREACH OF THE EXPRESS LIMITED WARRANTY ON THE PRODUCT OR ANY WARRANTY IMPLIED BY OPERATION OF LAW. OTHER THAN THE EXPRESS LIMITED WARRANTY SET FORTH ABOVE, THERE ARE NO OTHER WARRANTIES THAT ACCOMPANY THIS PRODUCT AND ANY ORAL, WRITTEN OR ANY OTHER REPRESENTATION OF ANY KIND IS HEREBY DISCLAIMED.

Non-Applicability of Warranty

In addition to any other limitations on the warranty discussed above, the warranty shall expire and be of no force or effect upon the occurrence of any of the following:

- (i) the cause of damage to the Product is due to improper handling or use.
- (ii) the Product is not properly maintained by the First User as determined by Ameda.
- (iii) non-Ameda approved accessories or spare parts are used with the Product as determined by Ameda.

If any of the above occurs, Ameda shall not be responsible for damage to the Product, parts thereof or injuries arising therefrom, either directly or indirectly. The warranty terms set forth above replaces all prior warranties with respect to the purchased Product, whether in writing or otherwise.

Outside the United States: Call your local distributor or location where you purchased the Product. For a listing of distributors in your country, please visit www.ameda.com.

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©2013 Ameda, Inc.
HygieniKit and associated logos are trademarks of
Mothers Choice Products Ltd.
Ameda, Inc.