Harmony™

EN Instructions for use | ES Instrucciones de uso | FR Mode d'emploi

PLEASE READ AND SAVE THESE INSTRUCTIONS. FOR MOST CURRENT BREAST PUMP INSTRUCTIONS VISIT WWW.MEDELABREASTFEEDINGUS.COM

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Important Safeguards

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

The warning symbol identifies all instructions that are important to safety. Failure to observe these instructions can lead to injury or damage to the breast pump. When used in conjunction with the following words, the warning symbols stand for:

-  **WARNING** Can lead to serious injury or death.
-  **CAUTION** Can lead to minor injury.
-  **NOTE** Can lead to material damage.
-  **INFO** Useful or important information that is not related to safety.

WARNING: To avoid health risk and reduce the risk of injury:

- This product is intended for use by a single user only and should not be shared between users.
- Do not drive while pumping.
- Never use while pregnant, as pumping can induce labor.
- Clean and sanitize all parts that come in contact with your breast and breast milk prior to first use.
- Wash all parts that come in contact with your breast and breast milk after every use.
- Inspect all appropriate pump components before each use.
- Do NOT continue pumping for more than 2 consecutive pumping sessions if no results are achieved.
- Do not thaw frozen breast milk in a microwave or in a pan of boiling water.

IMPORTANT

- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the breast milk if bottles or components become damaged.

Take a moment to read through this entire instruction manual before using this product for the first time.

PLEASE SAVE THESE INSTRUCTIONS.

It is best to wait until your breastfeeding routine is established (approximately 4 weeks) before expressing breast milk, unless otherwise advised by your healthcare professional.

Glossary of Terms

2-Phase Expression® technology – research based technology that mimics a baby's natural nursing rhythm.

Expression Phase – slower sucking/pumping rhythm for gentle and efficient milk removal as quickly as possible.

Maximum Comfort Vacuum™ – highest vacuum setting where a mother still feels comfortable during pumping. It is different for every mother.

Stimulation Phase – fast sucking/pumping rhythm to stimulate the milk ejection reflex and to start the milk flowing.

Intended Use

For lactating women to express and collect milk from their breasts to complement breastfeeding.

If you have medical reasons or other needs for exclusively pumping, it is recommended that you use a hospital grade breast pump such as our Symphony Breast Pump. To learn more, visit us at www.medelabreastfeedingus.com.

Product Description

Harmony™ is a personal use manual breast pump that includes 2-Phase Expression technology and is capable of single pumping.

Table of Contents

Important Safeguards	2-3
Product Description	5
1. Cleaning Instructions	6-9
1.1 Before using for the first time	6
1.2 After each use.....	6-7
1.3 Sanitize daily	8-9
2. Assembly.....	10-12
2.1 Assembly of pump kit	10-12
2.2 Correctly fitting breast shields	13
3. Pumping	14-17
3.1 Device operation.....	14-15
3.2 Symphony Breast Pump Setup.....	16-17
4. Storing Breast Milk.....	18
5. Preparing & Feeding Breast Milk	19
5.1 Preparing breast milk	19
5.2 Feeding breast milk.....	19
6. Troubleshooting.....	20
7. Supplemental Information/Disposal.....	21
8. Warranty.....	22

Product Description

1x Handle

Item 8007170

1x PersonalFit™

Breast Shield (24 mm)

(for other breast shield sizes see **Section 2.2**, page 13)

Item 87073

1x Pump Body

Item 8007196

Diaphragm, Stem, O-Ring Assembly

Item 8207234

1x Valve

Item 87089

1x Membrane

Item 87088

2x Bottles

Item 87130

2x Lids

Item 87130

1x Bottle Stand


Item 8100462

1. Cleaning

1.1 Before using for the first time

Clean and sanitize the parts below before using your pump for the first time. Follow the cleaning and sanitizing instructions in **Sections 1.2 and 1.3** that follow.

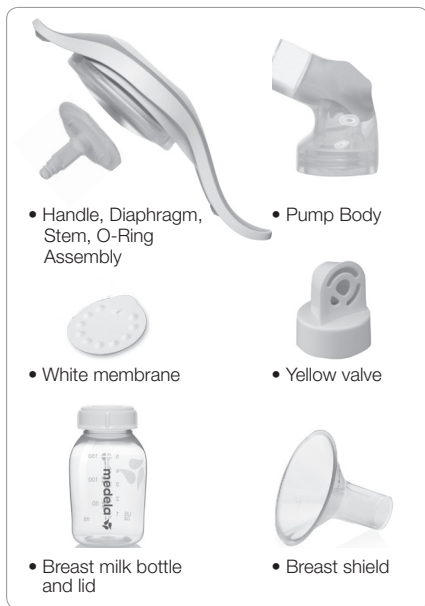1.2 After each use

Clean all parts that come in contact with your breast and breast milk.

Separate breast shield from pump body.

Rinse in cool water all separated parts that came in contact with breast and breast milk in order to remove breast milk residue.

Caution

- Only use drinking-quality tap or bottled water for cleaning.
- Take apart and wash all parts that come in contact with the breast and breast milk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.
- Pay special attention to O-Ring during disassembly and cleaning to avoid losing or damaging.

Supplies Needed:


- Mild Dish Soap
- Clean Dish Towel or Soft Brush
- Clean Sink or Bowl
- Drinking Quality Water

2

Twist and pull the yellow valve off the pump body.

3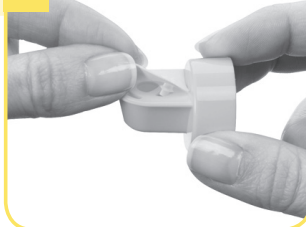

Remove the white membrane from the yellow valve.

Washing Options:

Washing in sink:

5

- Soak all separated parts in warm soapy water for 5 minutes.
- Clean with a clean dish-cloth or soft brush.
- Rinse all separated parts with clear water.
- Allow all parts to air dry in a clean area.
- Store dry parts when not in use.

or

- You may wash your kit parts by following the instructions on the Quick Clean™ Breastmilk Removal Soap bottle. (sold separately)

Washing in dishwasher:

- Wash all separated parts on top rack of dishwasher.
- Allow all breast pump parts to air dry in a clean area.
- Store dry parts when not in use.


1. Cleaning (cont.)

Note

- **Wash** breast pump kit parts after each use.
- **Sanitize** breast pump kit parts once daily.
- Take care not to damage parts of the pump kit during cleaning.
- If the individual components of the pump kit are cleaned in a dishwasher, parts may be discolored by food pigments. This has no impact on the function.
- The diaphragm, stem and o-ring assembly should not be sanitized using the Micro-Steam bags.

Diaphragm O-Ring Stem

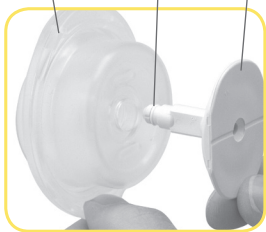1.3 Sanitize daily

Wash hands thoroughly.

3

Allow water to cool and gently remove parts from water with tongs.

1

Refer to image in Section 1.1

2

Separate all parts that come in contact with your breast and breast milk.

- Fill a pot with enough water to cover all parts.
- Bring water to a boil.
- Place parts in boiling water and let parts boil for 10 minutes.

4

or

- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts when not in use. Do NOT store wet or damp parts.

You may sanitize your kit parts by following the instructions on Medela's Quick Clean™ Micro-Steam™ bag. (sold separately) See note page 8. Do not microsteam diaphragm, stem, or o-ring.

2. Assembly

Parts you will need for this step:

2.1 Assembly of pump kit

Wash hands before touching breast pump, kit and breasts and avoid touching the inside of containers or lids.


Screw bottle onto pump body.

! Note

- Inspect valves for cracks or chips and inspect membranes for holes, tears or warping. If you notice any damage to these parts, replace them immediately.
- Check stem to make sure o-ring is present.

i Info


- Carry out all the steps with care and assemble the pump kit correctly for optimum performance.

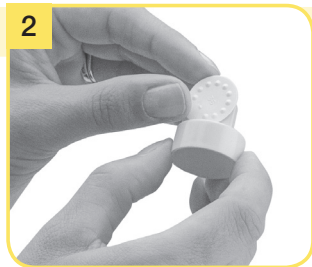
2 Snap membrane onto valve until membrane lies completely flat on valve.

3 Push yellow valve securely onto inner ring of pump body.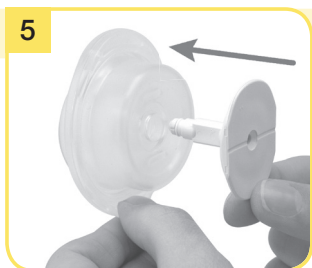

5 If the handle components have been disassembled, place the oval shaped base of the stem into the diaphragm.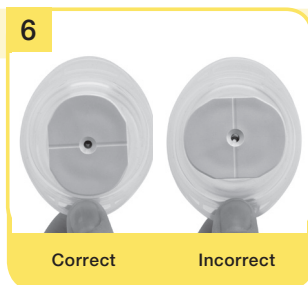

Continued next page.

6 **Hint:** Make sure the oval shape is lined up correctly in the diaphragm.


2. Assembly (cont.)

7


Place silicone diaphragm/stem into pump handle and press down firmly around entire edge to ensure proper seal. The lines on the diaphragm should line up with the yellow dots on the handle.

8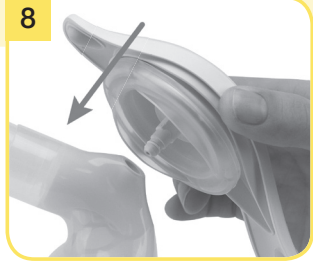

Insert stem of handle assembly into opening on top of pump body and push until you feel it snap in.

9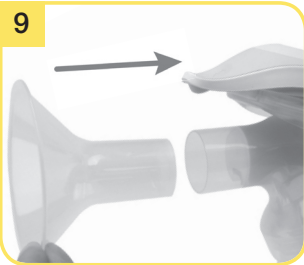

Insert PersonalFit™ breast shield into top opening on pump body.

2.2 Correctly fitting breast shields

1. Medela's PersonalFit sizing

Pumping should not hurt. For maximum comfort and pumping efficiency, Medela offers five breast shield sizes.

21 mm

24 mm

27 mm




30 mm

36 mm

Visit MedelaBreastshields.com to view the full guide to help you determine your optimal size based on your nipple diameter.

2. Test your breast shield size

1. Start with the 24 mm that came with your pump, or the size determined by measuring.
2. Center nipple and gently hold breast shield against your breast.
3. Adjust for Maximum Comfort Vacuum™, to achieve optimum suction level.
4. Refer to images while pumping in expression.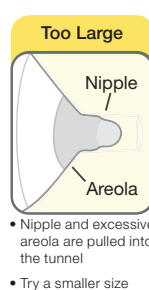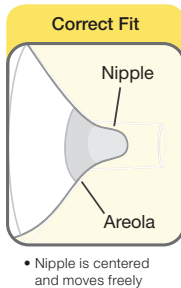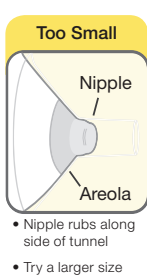

3. Should you try a new size?

- Does your nipple rub sides of tunnel, to the point of causing discomfort?
- Do you see excessive areola being pulled into tunnel?
- Do you see any redness?
- Is your nipple or areola turning white?
- Do you feel unexpressed milk after pumping?

If you answered **“YES”** to any of these questions, consider trying a new size by following the measuring instructions above.

If you are still unsure if you selected the correct size, see a lactation consultant, breastfeeding specialist, or visit MedelaBreastshields.com for assistance in choosing the right size breast shield.

Caution

- While some discomfort may be felt when first using a breast pump, using the breast pump should not cause pain.
- If you are experiencing discomfort at the base of the nipple due to rubbing of your breast tissue with the breast shield tunnel, use of a lubricant such as Tender Care™ Lanolin may be beneficial.
- For assistance with correct breast shield sizing and comfort please visit www.medelabreastshields.com or see a lactation consultant / breastfeeding specialist.

3. Pumping


Caution

- Wash hands thoroughly with soap and water before touching the breast and pump parts.
- Always inspect breast shields, valves and membranes prior to use for cleanliness. If dirty, see **Section 1.2**.
- Contact your healthcare professional or breastfeeding specialist if you can express only minimal or no milk or if expression is painful. See **Section 7** for more information.
- Do not try and express with vacuum that is too high and uncomfortable (painful). The pain, along with potential breast and nipple trauma may decrease milk output.
- **Step 2:** Do not hold the pump kit by the bottle. This can lead to blockage of the milk ducts and engorgement.

3.1 Device operation

1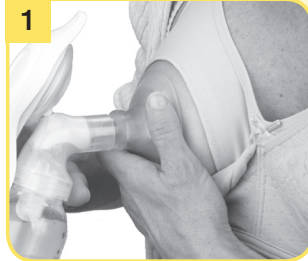

Center breast shield over nipple. Caution: pushing shield too firmly into breast may actually reduce milk flow.

3

Start pumping in the stimulation mode by pressing down on the milk drops symbol  on top of the pump handle. Press and release quickly until you see milk flow begin. This mode mimics baby's initial rapid sucking and is designed to stimulate milk flow/let-down.

2

The adjustable handle can be positioned for your maximum pumping comfort. The level of vacuum is controlled by the compression and the speed of the handle stroke. You decide the pumping rhythm that is best for you.

4

Once let-down has begun, switch to the expression mode by pressing the bottom of the pump handle. This will give you a longer stroke that mimics baby's deeper, more relaxed sucking and is designed to maximize milk flow.

3. Pumping (cont.)

i Info


- Requires membrane cap #8100769, membrane #8200094 and tubing #8007213. Sold separately.

3.2 Symphony Breastpump Setup

Remove handle assembly from Harmony™ breast pump.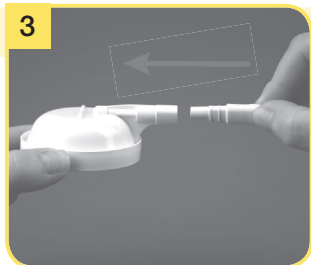

Insert yellow end of tubing into port on membrane cap.

1
Insert end of tubing into opening on top of Harmony pump.

2
Assemble protective membrane and membrane cap.

4
Push membrane cap firmly into place on pumping mechanism. Adjust for comfort once pumping begins.

4. Storing Breastmilk

Warning

- Do not thaw frozen breast milk in a microwave or in a pan of boiling water.
- Do not microwave breast milk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. (Microwaving can also change the composition of breast milk.)

4.1 Storing breast milk

- As soon as you are finished pumping, place the breast milk bottle or breast milk storage bag in a cooler with ice pack.
- Transfer breast milk bottle or breast milk storage bag to the refrigerator or freezer once you are home.
- When freezing, do not fill container more than 75% full to allow space for expansion. Label container with date of pumping.

5. Preparing and Feeding Breastmilk

Freshly Expressed Breastmilk Storage Guidelines (For Healthy Term Babies)				
Room Temperature	Cooler with Ice Pack	Refrigerator	Freezer	Thawed Breastmilk
4–6 hours at 66–78 °F (19–26 °C)	24 hours at 59 °F (15 °C)	3–8 days at 39 °F or lower (4 °C)	6–12 months 0–4 °F (-18–20 °C)	use within 24 hrs

References: www.BreastmilkGuidelines.com

5.1 Preparing breast milk

- Thaw breast milk overnight in the refrigerator. Thawed breast milk is safe in the refrigerator for 24 hours. Do not refreeze thawed breast milk.
- Quickly thaw breast milk by holding the bottle under warm running water.
- Place the sealed bottle in a bowl of warm water for 20 minutes to bring it to body temperature.
- If you are adding expressed breast milk to a container of already frozen breast milk, make sure to add a lesser amount than the already frozen amount.

5.2 Feeding breast milk

It is recommended that breastfeeding is well established prior to bottle feeding your baby.

- Always inspect the bottle, nipple and other components immediately before and after each use. If nipple appears cracked or torn, discontinue use immediately.
- To prevent possible choking hazard, test strength of nipple by pulling on bulb portion of the nipple.
- Do not attempt to enlarge the nipple hole.
- Infants must not be bottle fed without adult supervision.
- Nipple should not be used as a pacifier.

www.shopmedela.com

6. Troubleshooting

Low or No Suction	<p>Valve and Membrane Check</p> <ul style="list-style-type: none">• Remove and separate valve from membrane.• Inspect the valve for chips or cracks. Inspect the membrane for holes or tears. If flawed or damaged, replace valve and/or membrane prior to pumping.• Wash valve and membrane per Cleaning Section 1.• Reassemble. Make sure the membrane lays flat on the valve and if it does not, replace the valve and/or membrane prior to pumping. <p>Breastshield and Pump Body Check</p> <ul style="list-style-type: none">• Inspect breast shield and pump body to make sure there are no cracks and they securely attach to each other.• Make sure the stem assembly on the pump handle is seated all the way into the pump body until it will go no farther.• Make sure the white o-ring is in place in the stem handle.• Make sure the pump body is firmly screwed onto the bottle and that the breast shield is firmly seated into the pump body.• Make sure the oval base of the stem is properly seated in the oval base of the diaphragm.• Make sure the diaphragm is pushed all the way onto the circular rim on the underside of the handle.• Make sure the breast shield forms a complete seal around the breast.
No “let-down” or breast milk expression	<ul style="list-style-type: none">• Ensure your breast pump is assembled correctly and that there is suction.• Relax and take a 10-15 minute break if let down is not achieved after 2 consecutive pumping sessions.• Consult with your healthcare provider or breastfeeding specialist if expression does not occur.

7. Supplemental Information/Disposal

Supplemental Information

Following are some common breastfeeding related conditions. If you experience any of these symptoms, contact a healthcare professional or breastfeeding specialist.

	Symptom	Potential Causes
Engorgement	Breasts are hard and uncomfortable, possibly with reddened areas.	Milk not draining properly from breast. Milk can drain back into the tissue causing areas to swell and become tender.
Blocked/Plugged Ducts	Area of breast will look reddened and may be tender to touch.	Milk not being drained from a specific duct. The area becomes "clogged" up and milk is then prevented from flowing.
Mastitis	It often follows engorgement but can just occur suddenly. The initial symptoms are similar to flu – fatigue, headache and muscle aches, fever and localized breast tenderness. It can be in one or even both breasts and needs immediate treatment.	A bacterial infection in the breast tissue which is often accompanied by cracked nipples.

A healthcare professional or breastfeeding specialist can give you guidance in breastfeeding your baby and using a breast pump. Their guidance is valuable in terms of the lifetime of health benefits for both you and your baby. A listing of Lactation Professionals are available at www.medelalocator.com or call 1-800-TELL-YOU (1-800-835-5968).

If you have not resolved the problem with your breast pump or you have further questions, please contact Medela Customer Service at 1-800-435-8316 or email customer.service@medela.com.

Disposal Guideline

Properly dispose of the device in accordance with local, state and federal regulations.

8. Warranty

Caution

This is a single user product. Use by more than one person may present a health risk and performance questions and voids the warranty.

This product is warranted by Medela, Inc. to the original retail purchaser to be free from defects in material and workmanship for the period of 30 days (30 days for parts and accessories) from the date of purchase. In the event of a defect, Medela will repair or, at Medela's option, replace this product, without charge for such replacement, parts or labor. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product used commercially or which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO A DURATION OF 30 DAYS FROM DATE OF PURCHASE. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE.

Before you make a claim under this warranty, it may save time and expense to call Medela Customer Service (toll free) at 1-800-435-8316. You may also call this number for additional information concerning this warranty. If you wish to make a claim under this warranty, you must return this product to Medela with a return authorization number received from Medela Customer Service, prepaid, together with your dated bill of sale or other proof of purchase and a brief statement of the problem to the following address:

Medela, Inc. – Returns, Door 4501
1101 Corporate Dr.
McHenry, IL 60050
ATTENTION: RETURNS

Medela Canada, Inc.,
4160 Sladeview Crescent, Unit #8,
Mississauga, Ontario,
Canada L5L 0A1
ATTENTION: RETURNS

Call first for authorization number. Returns not accepted without an authorization number.

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Printed in the USA. Medela, 2-Phase Expression and Symphony are registered trademarks of Medela Holding, AG.
Quick Clean, Harmony, Micro-Steam, Maximum Comfort Vacuum and PersonalFit are trademarks of Medela, Inc.
Quick Clean, Harmony, Micro-Steam, Maximum Comfort Vacuum y PersonalFit son marcas comerciales de Medela, Inc.
Imprimé aux États-Unis. Medela et 2-Phase Expression sont des marques déposées de Medela Holding AG.
Swing, Quick Clean, Micro-Steam, Maximum Comfort Vacuum et PersonalFit sont des marques de Medela, Inc.